


Updating your Skyla Solution via USB

1. Using a suitable PC, open a web browser and navigate to <https://vepalabs.com/downloads/>
2. Once at the Downloads page, click the Skyla Solution icon to download the latest version of the Solution software.
3. If necessary, confirm that you wish to save the file, and wait for the download to complete. The file is about 700MB, and may take 5-10 minutes to download depending on your internet connection and speed.
4. Find a FAT32-formatted USB stick, or format a USB stick with the FAT32 filesystem (Call 1300 837 252 if you have USB format issues).
5. Locate the downloaded file and copy the '.bin' file onto the USB stick. The .bin file should be the only file on the USB. Delete other files if necessary

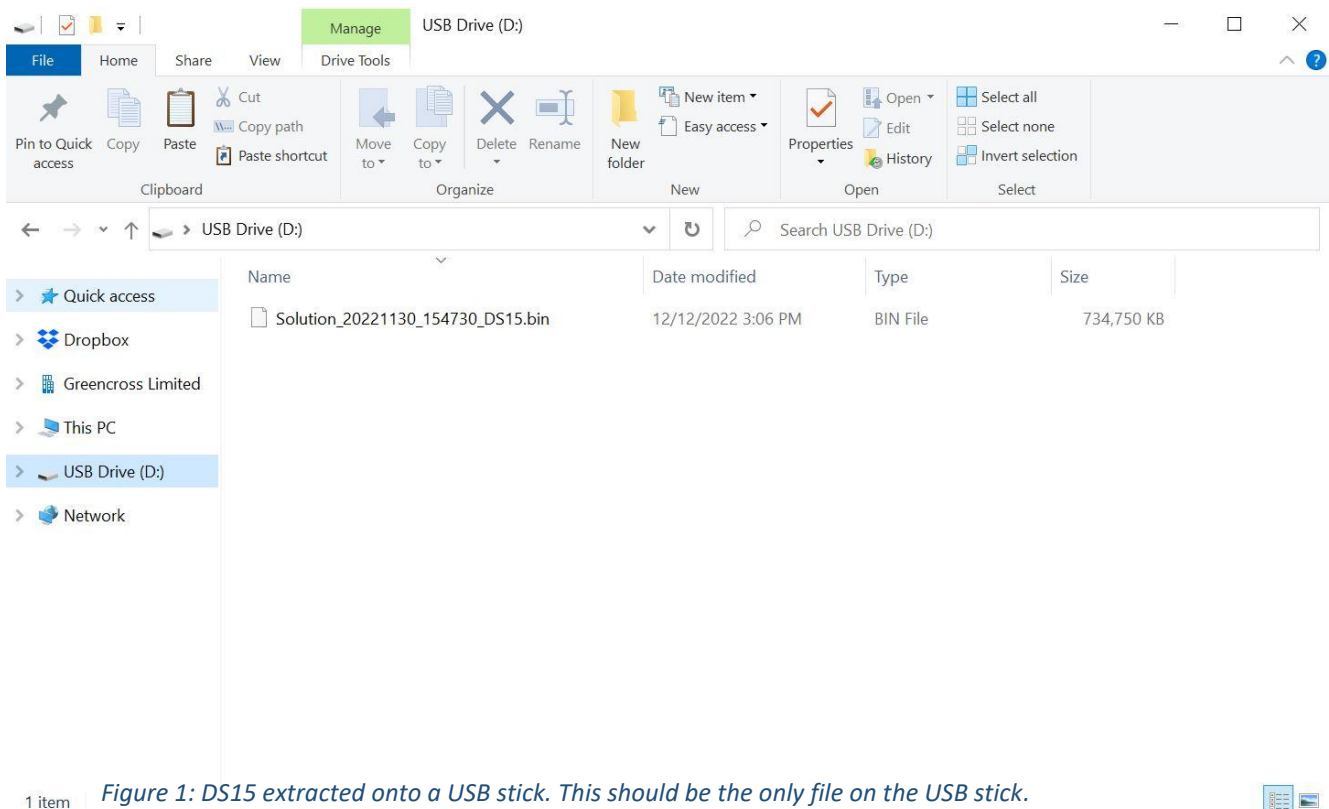

6. Remove the USB stick from the computer and insert into a USB port on the back of the analyser.

7. From the main screen, navigate to **Settings** -> **Administrator Settings** -> **System Upgrade**



Figure 2: System Upgrade button within Administrator Settings.

8. Press **'YES'**, then **'USB'** to confirm upgrade
9. No further input is required. Wait approximately 10-12 minutes while the file is copied to the analyser, installed, and the Solution restarts. Once complete, the analyser will return to the main screen.
10. Congratulations, you've just updated your Skyla Solution!

If you have queries or concerns, please contact Vepalabs on 1300 837 252 (Option 1).